INDICE - INDEX

Pagina - PAGE	Lingua - LANGUAGE
2	IT
3	EN
4	DE
5	ES
6	FR
7	CS
8	RU
9	BG
10	DA
11	EL
12	ET
13	FI
14	GA
15	HR
16	HU
17	LT
18	LV
19	MT
20	NL
21	NO
22	PL
23	PT
24	RO
25	SK
26	SL
27	SR
28	SV
29	TR
30	UK

IT

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EN

READ CAREFULLY BEFORE EACH USE OF THE BALL

DIRECTIONS AND WARNINGS

This document is the **essential guide** for a proper and safe usage of the **LEDRAGOMMA Original Pezzi® Soffball MAXAFE®** (Ø 15-22-26-30-40 cm.) : it contains the **necessary information** in order to use the product with **full satisfaction** and in the respect of the **safety rules** during the life of the product itself and/or its parts.

PROVIDED USAGES - LIMITS AND EXCLUSIONS

1. This ball has been designed and arranged to be used in various fields, such as sport, fitness, gymnastics but also in the medical and educational ones. It must not be used for different purposes than the ones stated by **Ledragomma**.
2. This ball can bear up to Ø cm.22-150 kg (330 lbs.); Ø cm.26-150 kg (330 lbs.), Ø cm.30-250 kg. (550 lbs), Ø cm.40-400 kg (880 lbs) weights.
3. **IT IS NOT A TOY** and, in any case, it's not suitable for children under 36 months : small parts included. Choking hazard.
4. **DO NOT** perform exercises with barbells, dumbbells or any kind of weights on the ball.

PACKAGE CONTENT

5. The package contains a ball, a yellow valve and two white plugs.

INFLATION

6. To inflate the ball, air mattresses pumps, bicycle pumps (only in this case use the yellow valve included in the package) or air compressors must be used. **DO NOT** inflate by mouth.

FIRST USAGE: MODE AND PROCEDURE TO FOLLOW

7. By first usage, proceed in accordance to the following directions:
 - 7.1) start to inflate the ball slowly, with the above mentioned tools, having a break sometimes
 - 7.2) don't exceed the 90% of the maximum diameter, indicated on the ball itself and/or on the packaging
 - 7.3) close the hole with one of the two provided white plugs (the second plug is a spare)
 - 7.4) wait 24 hours before inflating it to its maximum diameter and start to use it.

DIRECTIONS AND WARNINGS FOR THE FOLLOWING INFLATIONS

8. For the following inflations, never exceed the prescribed and indicated maximum diameter. If an air compressor is used, proceed slowly while approaching the prescribed maximum diameter. The plug can be removed using a coffee spoon.

PRECAUTIONS TO ADOPT BEFORE EACH USAGE

9. Before each use, check the surface of the ball and don't use it if it shows cuts, abrasions, holes or any other kind of damage. In all these cases, the safety of the ball is compromised and the ball itself could instantly tear, causing the user possible injuries.
10. Before each use of the ball, wash it with water and delicate toilet soap and remove the more obstinate stains with diluted alcohol.

USAGE TEMPERATURE

11. Caution: the ball must be kept at room temperature (18-20°C). In case of particularly low temperature, proceed with inflation only after letting the ball reach the above mentioned room temperature.

USED SURFACES

12. The ball must be used on soft surfaces, such as carpets, rugs, exercise mats. Never use the ball in presence of sharp objects or close to heating elements; when not in use, never leave the ball directly exposed to the sun.

REPAIR WARNINGS

13. If the ball shows holes or cuts, don't try to repair it: self made repairs invalidate the integrity of the product causing serious risks for the safety and health of the user. Only on the **Ledragomma Original Pezzi® Maxafe balls (flexton silpower® material- Pat.No.EP 1 409 088 B1 - Pat. No. US 7,144,354 B2)** small punctures or cuts up to 5 mm in length can be repaired using the special repair kit "Fixa-kit".

TRACEABILITY

14. A traceability code of 6 digits is marked on the ball; it indicates the month and year of production for this item. Please keep this code in the case of a claim, in order to identify the production time and the batch materials.

DISCLAIMER

15. Ledragomma Srl declines any liability for damages to persons, animals or objects or to the environment arising from the a non consistent use or maintenance to the precautions and directions indicated in this leaflet accompanying the product .

NOTE

16. Colours and decorations may vary according to manufacturer.

Made in Italy by: Ledragomma Srl - Zona Industriale C.I.P.A.F. - 33010 Osoppo (Ud) - Italy

Tel.+39 0432/986049 - Fax +39 0432/986255 - info@ledragomma.com - www.ledragomma.com

Manufactured under Quality System Certification ISO 9001:2008 - 13485:2003 "medical devices"

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